

Knee Brace Suspension Sleeve BY MOBILITY BRACES

The **Knee Brace Suspension Sleeve** is designed to aid in the proper placement, suspension and comfort of knee braces. Designed with dual application options, the Knee Brace Suspension Sleeve can be applied either under the brace or over the brace (based on patient preference) to reduce brace migration and improve patient comfort.

UNDER THE BRACE FITTING INSTRUCTIONS

1. Apply the hook-n-loop pull-tab to the top side of the base directly above the sewn-in hook on the bottom side (A). Apply so 1" of pull-tab hook lands on base.
2. Place the sleeve around the back of the calf. Align the non-hook edge of the sleeve with your tibia (shin bone) and hold in place (B). The top should sit just below the popliteal (crease behind the knee). Pull the side with the hook and pull-tab around the leg (B). Continue to wrap the pull-tab side around the leg (overlapping the side without the pull-tab) and continue to wrap it around the leg securely until it fastens down on itself via the hook-and-loop (C).
3. Once Suspension Sleeve is in place (D), follow normal application instructions for your knee brace - applying the brace over the top of the applied Suspension Sleeve. Knee brace condyle pads (found on the inside of the hinge) should rest just above the top edge of the Suspension Sleeve (E) which now acts as a lip that catches the condyle pad if the brace begins to migrate. The added circumferential thickness and additional layer(s) of compression around the calf, in conjunction with the tackier outer surface when compared to skin - will aid in brace fit, suspension and comfort. PROPERLY APPLIED FRONT VIEW (F).

OVER THE BRACE FITTING INSTRUCTIONS

1. Apply the hook-n-loop pull-tab to the logo'd side of the base on the opposite side of the sewn-in hook on the bottom (G). Apply so 1" of pull-tab hook lands on base.
2. Properly apply your knee brace (*instructions below use Mobility READY KNEE Brace*).
3. Apply suspension sleeve's sewn-in hook to the exposed loop material next to the lower D-ring (H) on the brace. Once suspension sleeve is fixed to the knee brace (I), pull the sleeve forward around the front of the leg (J) and then wrap it around the leg until the sleeve ends and the pull-tab hook lands back on the sleeve loop - locking it in place (K). PROPERLY APPLIED FRONT VIEW (L) AND BACK VIEW (M).

WARNINGS

- The Knee Suspension Sleeve is not designed to prevent injury.
- If you experience skin irritation, pain or discomfort while using this product, immediately remove and contact your local healthcare provider.

CARE INSTRUCTIONS

- Hand wash in warm water with mild soap - do not machine wash or dry.
- Lay flat or line dry only - do not expose to direct heat sources such as heaters, hair dryers, direct sunlight, etc.

WARRANTY

- Knee Suspension Sleeves are warranted to be free of material or workmanship defects.
- Any defective components may be replaced or credited at the discretion of Mobility Braces, LLC.
- Knee Suspension Sleeves are warranted against defects for one year from date of purchase.

PARTS INCLUDED

- One (1) base sleeve garment
- One (1) hook-n-loop pull-tab

UNDER THE BRACE FITTING



OVER THE BRACE FITTING

