

How to wear your READY BACK Brace



STEP 1

Open brace and place it in the center on the lower back.

1



BACK VIEW

STEP 2

Wrap left flap around the body. Place the right flap on top adjust, securing to fit.

2




FRONT VIEW


STEP 3

Pull the overlapping straps up and out achieve desired lift and support, secure the straps across the front surface.

3

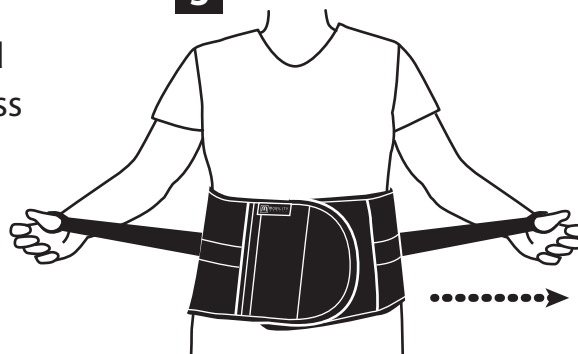

Fasten Across Front

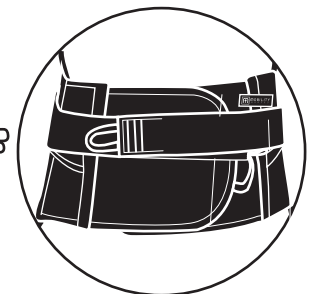

FRONT VIEW

Wear as directed by your physician.
Can be worn under or over clothing.



QUESTIONS:

If you have fitting issues call:
1-866-938-9162
Mon-Fri 8 a.m. to 5 p.m. MST



CLEANING INSTRUCTIONS:

If the brace has removable panels, take them out. Hand wash with mild soap and water. Lay brace flat and let air dry completely.
DO NOT machine wash or dry.